



**DEPPELER**

*Signature of excellence*

SHARPENING  
SOLUTIONS




## WHY CHOOSE INSTRUMENTS THAT CAN BE RESHARPENED?

A study tested and compared Deppeler instruments and instruments that cannot be resharpened. The results show that, after 1000 strokes on dentin and five reprocessing cycles, all instruments have lost about 75% of their effectiveness.

At this stage, if the practitioner uses an instrument that cannot be resharpened, the blunt instrument will continue losing its efficiency.

If the practitioner uses a Deppeler instrument, the cutting edge can be **resharpened to regain optimal sharpness and comfort.**

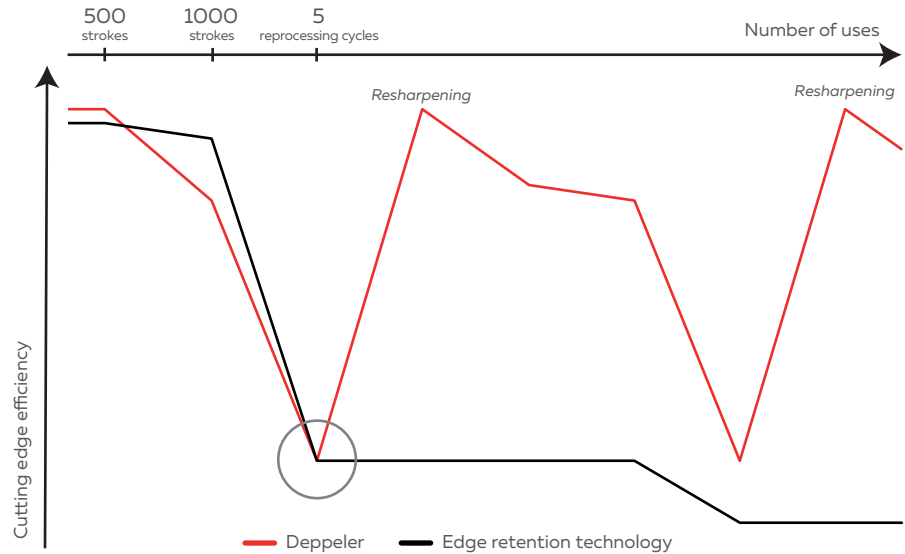

Schmidlin et. al.: In vitro evaluation of 3 curettes with edge retention technology after extended use. SSO Journal 119: 1200-1207. (12/2009)

### Flexibility

The working parts of resharpenable instruments are more flexible and less likely to break in the patient mouth.

### Efficiency

A bare steel edge is more efficient than a coated edge.

### Savings

Resharpenable instruments need to be replaced less frequently. Their cost is lower in the long run.

### Sustainability

In addition to having to purchase instruments less frequently, all-steel instruments are also easier to recycle.

## SHARPENING METHODS

Maintain the cutting edges of your scalers and curettes until the end of their life cycle. With Deppeler's foolproof Easy Sharp solutions, sharpen your instruments easily and effectively, without any facets, serrated edges, or burrs.




### MANUALLY

*Natural stones*



**Arkansas Hard Black**  
ref. ESS1  
For regular sharpening

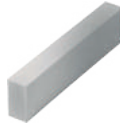

**Arkansas Translucent**  
ref. ESS3A  
Finest quality  
For faster sharpening and better results



**Silicone Carbide**  
ref. ESS4  
For titanium instruments and very blunt steel instruments



### WITH A COACH

**Easy Sharp Coach**  
ref. ESC

Designed for visual guidance and easy control of the sharpening angle.  
1 coach for universal curettes and scalers, 1 coach specifically for Gracey curettes.  
Suitable for all brands.

How to use an Easy Sharp Coach?



### WITH A MACHINE

**Easy Sharp Device**  
ref. ESD

Designed for maximum precision of the sharpening angle and optimal control of the movement.  
The cutting edge feels brand-new, back to its original state.  
Suitable for Deppeler instruments.

How to use an Easy Sharp Device?



# Order information

## EASY SHARP COACH

### ref. ESC

includes:

- 2 coaches (1 universal and 1 Gracey)
- 1 Arkansas Hard Black stone ESS1

### ref. ESCPLUS

includes:

- 2 coaches (1 universal and 1 Gracey)
- 1 Arkansas translucent stone ESS3A

## EASY SHARP DEVICE

### ref. ESD

includes:

- Device
- 1 Arkansas Hard Black stone ESS1

### ref. ESDPLUS

includes:

- Device
- 1 Arkansas translucent stone ESS3A

We recommend resharpener your instruments  
at least once every 5 to 8 treatments.



WWW.DEPPERLER.CH

f DEPPELERSA | @ DEPPELER\_INSTRUMENTS



# DEPPERLER

*Signature of excellence*